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Great care has been taken in compiling texts and illustrations for these instructions for use. Nevertheless, this does not exclude the possibility of typing errors or other erroneous information. Please note that DeguDent GmbH bears no liability for these.

Release date: 02/2010

Instructions for use when working with the DeguDent Compartis® Center

Thank you for your interest in Compartis® integrated systems. Please read the following instructions carefully to ensure that the product is to your complete satisfaction.

General information

Proper connection

The Cercon® eye is installed in your laboratory by a DeguDent technician. He will ensure that the scan module functions properly and that the Cercon eye can be linked to the Compartis Production Center.

Selecting the Compartis button in the Cercon® art menu allows you to decide on the order submission. Should you decide to go ahead with the order, the CAD data is sent to the Compartis Production Center via your own DSL/ADSL connection.

Specific information

Compartis services

Compartis services are to be introduced in stages and will provide you with frameworks and “special designs” (such as abutments) for a wide range of indications.

We are expanding our Compartis product range by continuously adapting materials and restoration types to your needs.

See the instructions for use of Compartis crown and bridge frameworks for updated indications / contraindications for Compartis restorations

Please check our web-site www.compartis.net for the current range of Compartis products.

Model fabrication

Model fabrication

The cameras of the Cercon® eye scan module can record the surfaces of dies or jaw segments on virtually any model fabricated with pastel-colored dental stone. Esthetic-base® gold (Dentona AG, Dortmund) proved particularly suitable for scanning.

If it is necessary to block out the scan model, we recommend you use our Cercon® eye undercut-wax. Please use a commercially available scan spray (Cercon® art Scan Spray) when scanning less suitable, i.e. dark-colored dental stone models.

Die preparation

Cercon eye recognizes

- **Shoulderless preparations**
(for metal frameworks)
- **Deep chamfer preparations**
(for porcelain and metal frameworks)
- **Shoulder preparations**
(for porcelain and metal frameworks)

Please note that all preparations should be clearly delimited at their apical end using a slight chamfer shape (no deeper than 30° below the preparation margin) which tapers off evenly toward the die. We recommend preparing the die with a round bur (Diameter: 2.3 mm / 0.09 in) or a cutter (Diameter: 2.3 mm / 0.09 in).

Over-pronounced U-shaped preparations cannot be captured by Cercon eye and lead to incorrect scan results.

In this case please correct the preparation as required.

Model alignment

After preparation of the model section to be scanned, place the model in the model holder and align the height of the model using the scan template. Adjust the height and angle of the model holder so that all relevant dies are in simultaneous contact with the underside of the template.

Secure the table of the model holder by turning the hydraulic screw, which stops any movement of the table.

Then remove all model segments, except the die to be scanned, from the base (this avoids “shadows” caused by

the adjacent segments which can detrimentally affect scanning) and place the model table on the Cercon eye rotary unit. Select the preview using the Cercon art button “Video” and align the die in the centre of the cross-hairs. Then close the cover of the Cercon eye to start the scan procedure.

When scanning dies for bridge frameworks, please scan them in the sequence stipulated by Cercon art. Ensure that the three reference marks of the model holder are clearly visible in the preview of each scan. It is also important to ensure that the fixed position of the model holder does not alter when repositioning the dies and jaw segments.

The reference marks on the model holder must always be kept clean.
Avoid scratching or defacing the reference marks when cleaning the model table!

Subsequent working stages are described in detail in the Cercon art 3.0.3. Instructions for Use.

Delivery

After taking delivery of your Compartis order, check the goods received against the numbers listed on the delivery sheet. If you have sent us different orders on the same day, you will normally receive the frameworks as one delivery.

After you have checked the frameworks for visual damages such as cracks or chipping, follow the steps below:

The crown and bridge frameworks you receive from us have been fabricated according to the data you have sent us. If you have a complaint about a Compartis framework, you have 14 days after receipt of goods to contact us using the complaints procedure implemented in Cercon art.

The procedure is as follows:

Put a cross in the "Revision" box in the selection window "Open existing file". The list then reduces to cases which have already been sent to Compartis. When the relevant case is opened, a form comes up in which you select at least one description relating to your complaint. If none of the points listed describes your complaint, then please write the reason for your complaint in the text box.

If the reason for complaint is a damaged frame, just send the case without a new scan. If you are dissatisfied with the fit or marginal seal, then put a cross in the box "new scan" on the form.

The subsequent procedure takes you automatically into the scan section. With the new scan please ensure that all relevant die areas are properly scanned, in particular those areas in which there were discrepancies.

Then send the case to the Compartis network production.

No alterations can be made to the input for a remake except for a new scan. Only previous cases can be opened for remake.

In the case of a valid complaint you will receive a newly fabricated framework within 72 hours free of charge.

Standard Terms and Conditions for Compartis integrated systems® of DeguDent GmbH

Small print

Working together with us should be as straightforward as possible. We have included ordering terms and conditions to ensure that this is also the case in critical situations.

When sending us an order you accept the ordering terms and conditions listed below.

1. General Provisions

The following Standard Terms and Conditions for Compartis integrated systems® shall apply only towards enterprises (Section 14 German Civil Code). They form an integral part of all current and future contracts and agreements within the scope of our Compartis product range concluded with us. In particular, they apply to sales, work and service contracts within this product range. The Purchaser acknowledges the applicability of these Terms and Conditions – also for follow-up business, if any – by not objecting to them upon receipt, at the latest upon receipt of our goods or other services. Oral side agreements, exclusions from, amendments of or additions to these Standard Terms and Conditions and to the contracts concluded on the basis hereof require express written confirmation by us to be valid. This shall also apply to any waiver of this written form requirement. The applicability of other general terms and conditions of the Purchaser is hereby objected to, even for the event that they are submitted to us in a confirmation letter or in any other way. Any unconditional delivery of goods and services as well as the acceptance of payments on our part does not constitute an acknowledgement of any provisions deviating from these Standard Terms and Conditions. Unless otherwise specified in these Standard Terms and Conditions, the terms and definitions of INCOTERMS 2000 shall apply.

2. Orders, Offers, Contracts

Orders of the Purchaser are binding; they can not be revoked as soon as we received them. Our offers are subject to confirmation; a contract is only established by our written order confirmation or if orders have been executed by us. Statements and notifications by the Purchaser subsequent to the conclusion of the contract are only valid if made in writing.

3. Prices, Terms of Payment, Set-off

All prices are "ex works", excluding packaging, plus applicable statutory VAT. Payment is due immediately upon invoicing and shall be made to us free payment office, unless otherwise agreed. Deductions of discounts must be agreed separately in writing. The Purchaser may only offset such claims which are uncontested or have become non-appealable.

4. Place of Performance, Delivery, Duty to Cooperate

The place of performance is our place of business. Unless otherwise provided for in the order confirmation, delivery is agreed to be "ex works". The risk passes to the Purchaser when the products are shipped, even when freight is paid. The scope of our delivery obligation is exclusively defined in the contract concluded with us. Our goods are subject to changes in construction, form and colour resulting from technical improvements or legal requirements, provided such changes are not substantial or otherwise unacceptable for the Purchaser. If the Purchaser can be reasonably expected to accept partial deliveries, these may be made and invoiced. Terms of delivery are generally stated subject to the cooperation of the Purchaser in accordance with the contract. The compliance with our delivery obligation requires the timely and due fulfilment of the Purchaser's obligations. If after conclusion of the contract it turns out that the Purchaser does not sufficiently guarantee its solvency, and our claim for payment is endangered, we may withhold delivery until the Purchaser has effected payment or furnished a security. If payment or security has not been received within 12 working days of request, we are entitled to withdraw from the contract. If the Purchaser defaults in calling, taking delivery of, or collecting goods, or if the Purchaser is responsible for a default in shipment or delivery we shall be entitled, without prejudice to any further claims, to charge a flat charge in the amount of locally customary storage costs, irrespective of whether we store the goods in our warehouse or with a third party. The Purchaser may prove that no damage has occurred or that the amount of damage was lower.

5. Delivery Dates, Default in Delivery

Delivery dates, delivery periods and performance periods are agreed in writing. The date of the order confirmation is authoritative for the commencement of such periods. If we are unable to comply with the agreed delivery date for circumstances on our part or on the part of our suppliers which are not within our control, the delivery date shall be reasonably extended. We will promptly inform the Purchaser of such circumstances. If the impeding circumstances still exist one month after expiry of the agreed delivery date, either party may withdraw from the contract. Further claims for default in delivery, for which we are not responsible, shall be excluded. If we do not receive delivery despite having placed equal orders with reliable suppliers, we will be released from our performance obligation and may withdraw from the contract. In the event of a default in delivery, the Purchaser shall in case of proven damage be entitled to claim a voucher for a free Compartis replacement of the same type and the same material (flat compensation for delay). The Purchaser can furthermore grant a reasonable additional period, which may not be less than 15 working days. If the additional period expires without result, the Purchaser is entitled to withdraw from the contract or to claim damages instead of performance. Any claims for damages due to a default in delivery, and claims for damages instead of performance, which exceed the aforementioned flat compensation, shall be excluded in all events of delayed delivery, including after expiry of the additional period of delivery. The limitation of liability shall not apply if the default is caused by intent, gross negligence or substantial breach of duty. It does also not apply if a commercial transaction for delivery by a fixed date has been agreed. The foregoing provisions do not entail any change of the burden of proof to the prejudice of the Purchaser.

6. Retention of Title

We retain title to the delivered goods until all our receivables against the Purchaser from the business relationship have been settled, including future receivables from contracts concluded at the same time or later. This shall also apply if receivables are billed periodically and the balance is brought forward and accepted. The Purchaser is entitled to sell or process the goods in the ordinary course of business. Should processing be required, the Purchaser shall do it for us; we do, however, not incur any ensuing obligation. In the case of processing, connecting or mingling of retained goods with other items, we shall generally acquire co-ownership in the newly created product, i.e. in the event of processing at a ratio of the invoiced value (= invoiced gross value including incidental charges and taxes) of the retained goods at the value of the newly created product, in the event of connecting or mingling at a ratio of the retained goods in proportion to the invoiced value of the other items. The Purchaser hereby assigns to us all claims arising against its customers or third parties from the resale. The Purchaser remains authorised to collect such claims also after the assignment. Our entitlement to collect claims ourselves shall remain unaffected thereby; we shall, however, not make use of such right as long as the purchaser duly complies with its payment and other obligations. Upon our request, the Purchaser shall inform us about assigned claims and the respective debtors, give us all necessary information for collection, hand over the relating documents and inform the debtor of the assignment. If the Purchaser acts in breach of contract, in particular in the event of default in payment, we are entitled to withdraw from the contract and take back the goods. For the purpose of taking back the goods, the Purchaser hereby irrevocably grants us access to its business and warehouse premises in order to collect the goods.

Within the scope and period of our retention of title, the Purchaser may neither assign nor pledge any goods or products produced from such goods without our consent. The conclusion of financing agreements (e.g. leasing agreements), which include the assignment of our retention rights, are subject to our prior written consent, unless the contract obligates the financing institution to pay the part of the purchase price owed to us directly to us. Purchaser must immediately inform us in writing about attachments or other interventions by third parties. Purchaser is prohibited from making arrangements with its customers which might interfere with our rights. We undertake to release collateral pledged to us, if Purchaser so desires and at our own discretion, provided that the realisable value of the collaterals exceeds the receivables to be collateralised by more than 20 % or if it exceeds their nominal value by more than 50 %.

7. Product Information

Our statements on our products as well as about our systems and procedures are based on extensive research and technical application experience. We provide these results, with which we do not assume any liability exceeding the respective individual contract, in word and writing to the best of our knowledge, but reserve the right to technical amendments within the scope of product development. This, however does not release the Purchaser from personally examining our products and procedure for applicability for own use. Application specifications of the Purchaser are only relevant if we confirmed to the Purchaser in writing upon conclusion of the contract that the delivered products are suitable for the use intended by the Purchaser. This also applies with regard to the safeguarding of proprietary rights of third parties as well as for applications and procedures.

8. Acceptance

Because no formal inspection is carried out the goods shall be deemed to be accepted with use of the goods by the Purchaser, at the latest with expiry of one week after delivery, if the acceptance has not been rejected. The acceptance may not be denied due to minor defects, which do not significantly impair the serviceability.

9. Defects of Quality

Claims of the Purchaser for defects of products are statute-barred within one year from delivery or installation of the agreed item of delivery and performance, unless otherwise agreed. In the event of fraudulent concealment of defects, wilful conduct or the assumption of a quality guarantee, the statutory limitation periods shall apply. The statutory limitation periods shall also apply to recourse claims under purchase contracts for consumer goods of the Purchaser or its customers.

The Purchaser is obliged to personally, or through the recipient named by it as its vicarious agent, examine the goods or performance without undue delay after delivery. With regard to the examination obligation and obligation to notify defects, Section 377 German Commercial Code applies. Obvious defects – also deviations from any agreed quality – have to be notified in writing without undue delay, hidden defects directly upon detection, at the latest however within the limitation period mentioned in para. 1 above. If the Purchaser fails to notify any defect in due form and time, the goods are considered to be approved.

The Purchaser may not derive any further rights from defects of quality which do not or only insubstantially impede the value and the suitability of the goods for the use noticeable to us. In the case of justified complaints, we are at our choice obliged to either delivery faultless replacements or to remedy the defect without charge, unless the expenses arising from the subsequent performance are disproportionate in comparison with the defect and are unacceptable for us.

The Purchaser is obliged to re-assign any replaced or exchanged parts to us.

If the subsequent performance fails, the Purchaser shall at its choice be entitled to withdraw from the contract or to request reduction of the purchase price. If according to legal regulations we are obliged - irrespective of the cause in law - to pay damages based on a defect, this obligation to pay damages shall be limited according to clause 10 below.

10. Liability

Any liability for damages exceeding the provisions set forth in clause 9 shall - irrespective of the legal nature of the asserted claim - be excluded. This in particular applies to claims for damages from culpa in contrahendo, for other breaches of duty or claims under the law of torts for compensation of damage to property according to Section 823 German Civil Code. This does not apply insofar as liability is mandatory, e.g. according to the Product Liability Act, in any case of intent, gross negligence and due to the violation of essential contractual obligations. The claim for damages for violation of essential contractual obligations, however, shall be restricted to the foreseeable damage typical for this kind of contract, unless liability is based on intent or gross negligence or harm to life, body or health. The foregoing provisions do not entail any change of the burden of proof to the prejudice of the Purchaser. Insofar as the liability for damages is excluded or restricted in respect of us, this shall also apply to the personal liability for damages of our employees, staff members, assistants, representatives and vicarious agents. Insofar as the Purchaser is entitled to claims for damages according to this Clause 10, these shall fall under the statute of limitations upon expiry of the limitation period applicable to claims for defects of title according to Clause 9. In the event of claims for damages according to the Product Liability Law, the legal statute of limitations regulations shall apply.

11. Statute of Limitations

Claims of the Purchaser other than claims based of defects shall be statute-barred within two years from statutory commencement of the statutory limitation period. The foregoing limitation does not apply to our liability arising from intent and gross negligence, tort, the Product Liability Act, and in the case of culpable injury to body, life and health.

12. Statements of Account

The Purchaser is obliged to review statements of account, in particular balance confirmations as well as other accounts and notices for correctness and completeness. Objections against statements of account have to be submitted within one month from receipt; other objections shall be raised without undue delay. The omission of timely objections is considered as approval; statutory claims of the Purchaser for justified objections after the expiry of the period shall remain unaffected.

13. Place of Jurisdiction

If the Purchaser is a merchant, the place of jurisdiction shall be our registered office. However, we are also entitled to take action against the Purchaser at its general place of jurisdiction.

14. Governing Law

All legal relationships between the Purchaser and us shall be governed by the laws of the Federal Republic of Germany, to the exclusion of the UN Convention on the International Sale of Goods (CISG).

15. Partial Invalidity

Should individual provisions of these Terms and Conditions be or become invalid as a whole or in part, the validity of the remaining provisions shall remain unaffected thereby.

Note:

We regularly store and process the Purchasers' data by EDP to the extent required for proper handling of the contractual relationships.

Our Customers' and Purchasers' data is regularly stored and processed by EDP to the extent required for proper handling of the contractual relationships.

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A Dentsply International Inc.