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Solaris Goldbad

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Instructions for Use

Solaris gold solution

The **Solaris gold solution** is a non-cyanide gold sulphite solution. By adding Solaris Activator (as per dosage instructions), galvanic precipitation (electroplating) can be achieved for dental use (for inlays, crowns and bridges).

Dosage:

The amount of gold solution used will depend on the size of the objects to be electroplated. The power settings and layer thicknesses are set on the device using the Solaris model for parameter determination. The Solaris unit will automatically calculate the amount of liquid needed.

Amount of liquid	500 ml
Gold content (net usable)	6.75 g
Working temperature	Around 63 °C
Colour of liquid	Light yellow
Colour of precipitation	saturated yellow, high lustre
Time for precipitation/layer thickness	5 h/200 µm – 6 h/250 µm – 7 h/300 µm
Movement in bath	Mandatory

Instruction for processing:

⚠ **Attention:** The **Solaris gold solution** can be used only once. Use original Solaris Activator only.

Avoid impurification and mixing the gold solution with other materials. They will affect the electroplating result negatively or even make the solution unuseable. Any objects dropped in the bath must be removed with plastic tweezers only. To avoid contamination of the gold solution, close the bottles immediately after use and store in a cool place, well protected from sunlight.

Bath liquid quantities and activator dosage:

The layer thickness (µm) and the surface of the objects to be electroplated will determine the amount of gold bath to be used. The power settings are determined by the Solaris model for parameter determination. Once the parameters have been set on the unit, the amount of liquid required for the process is displayed.

⚠ **Attention:** Shake Solaris Activator well before each use and check for crystal formation. If crystalline particles are visible in the activator, do not continue to use it.

Solaris Activator is added to the gold solution at a ratio of 1:20 (e.g. 5 ml of activator is added to 100 ml of gold solution). The quality of the precipitation depends on the exact dosage of the liquid and the activator and the correct power settings on the device.

Recycling:

Effected by precipitation cathode. See instructions for use.

Protective measures:

The gold solution may soil and stain surfaces and clothes. Use proper precautions as usual when handling chemicals.

Storage:

Avoid contamination, direct sunlight and extended exposure to oxygen. After use, keep container tightly closed and in a well-ventilated place, well protected from sunlight.

Shelf life:

See label.

Disposal:

Any product residue must be disposed of in accordance with applicable laws and regulations.