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Prestomat[®]

Printer

Instructions for use
Mode d'emploi
Istruzioni per l'uso
Instrucciones para el uso

Prestomat® Printer

D	Gebrauchsanweisung	5
GB	Instructions for use	11
F	Mode d'emploi	17
I	Istruzioni per l'uso	23
E	Instrucciones para el uso	29

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Table of contents

1	Scope of delivery	12
2	Accessories available	12
3	Technical data	12
4	General information	12
5	Inserting and exchanging the paper roll	13
6	Mounting of the printer	13
7	Programming the casting log	14
	7.1 Setting of date, time and language	14
	7.2 Entering the laboratory name	15
8	Exchanging the ribbon	16

1 Scope of delivery

The scope of delivery of the Prestomat® *Printer* (REF 5522 0200) includes:

- 1 printer module
- 1 connecting cable
- 1 ribbon
- 1 paper roll

2 Accessories available

Spare ribbon	REF 2351 1404
Spare paper roll	REF 2351 1405

3 Technical data

Power supply:	5 V (via Prestomat compact Type 2)
Power consumption:	max. 6 W
Environment temperatures:	4 °C to 40 °C
Storage temperatures:	-10 °C to 50 °C
Dimensions (h × w × d):	78 × 118 × 230 mm
Weight:	approx. 15 N

4 General information

Read the operating instructions carefully before installing and operating the printer.

The information contained in these operating instructions must be adhered to!

The Prestomat® *Printer* must only be operated with original spare parts and accessories.

Only in this way will the performance data given and the required operational reliability be ensured.

Protect the printer against moisture to avoid short-circuits and damage to the printer.

Caution:

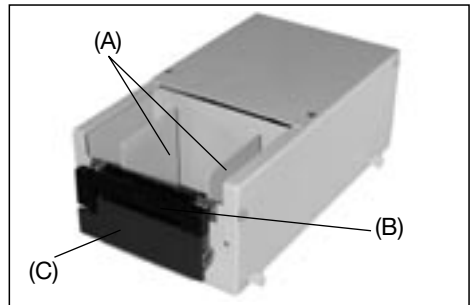
Power is supplied to the printer via the serial port.

Therefore do not connect any commercially available printer to the Prestomat® *compact*.

This will result in damage to the units!

5 Inserting and exchanging the paper roll

- Connect the printer at the port to the port on the right side of the housing of the Prestomat using the enclosed connecting cable.
- Turn the printer upside down to exchange the paper roll.
- Loosen the two screws at the front of the printer and remove the housing element.
- Place the paper roll between the two (A) guiding plates and thread the beginning of the paper into the holding slot(B).
- When inserting, ensure that the beginning of the paper is cleanly cut and observe the proper unwinding direction of the paper roll (paper is fed from the top).

- Switch the Prestomat on and press the paper feed key (C) until the beginning of the paper is transported beyond the tear-off edge.
- Close the printer housing.

6 Mounting of the printer

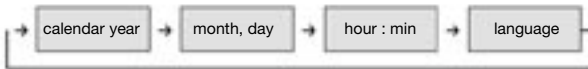
- The printer is mounted into the respective openings at the right side of the housing and fastened with the screw located at the printer.
- The printer is ready.

7 Programming the casting log

In addition to casting parameters such as casting temperature, time of further heating and pressure delay, the date, time and the laboratory name can be stored individually. Additionally you can select to print out the casting log in German, English, French, Italian and Spanish.

7.1 Setting of date, time and language

The settings are performed according to the following steps:

- Switch the Prestomat on, press the program key >Prog.< and then >Heizen (heat)<. The display will show the year.
- The correct year is set with the keys ▼▲.
- By pressing the key >E< the set year will be stored and the program step to set the day and the month is displayed. Please note that the month will be displayed first and then the day (e.g. 7.1 = 1. July).
- The correct date is entered with the keys ▼▲. To change to a new month either ▼ or ▲ is pressed until the new month is displayed.
- By pressing the key >E< the set date is stored and the next program step to set the time is displayed.
- Setting of the correct time is carried out by pressing the keys ts.
- By pressing the key >E< the set date will be stored and the program step to set the desired language will be displayed (1 = English, 2 = German, 3 = French, 4 = Italian, 5 = Spanish).
- The desired language is set with the keys ▼▲.
- By pressing the key >E< the desired language is stored. Programming is completed now. By pressing the program key >Prog.< again you can exit the programming mode.

7.2 Entering the laboratory name

A maximum of nineteen characters is available to enter the laboratory name. Each character resp. each letter features two-digit coding from 20 to BF which can be taken from the following table.

- Switch the Prestomat on and press the program key >Prog.< and then >Gießen (casting)<. The display shows a three-digit number. The first digit represents the position of the letter (e.g. 1**). The first letter can be programmed.
- Enter the code of the first letter by pressing ▼▲. Please note that the second digit represents the code number of the horizontal axis and the third digit the code number of the vertical axis. By pressing the key >E< the desired letter will be stored and the display returns to the second program position.

Example for entering the name DeguDent step by step:

D → 144 >E<
 e → 265 >E<
 g → 367 >E<
 u → 475 >E<
 D → 544 >E<
 e → 665 >E<
 n → 76E >E<
 t → 874 >E<

- Repeat the input until the complete name will be stored. If your laboratory name has fewer than 19 letters, enter a blank (space) until the display shows the code of the first letter of the laboratory name again. The code for blanks is 20 or 9F.

L\H	2	3	4	5	6	7	8	9	A	B
0	Ø	Θ	P	'	P	Ä	i	□	↓	
1	!	1	A	Q	a	q	Ö	N	...	↑
2	"	2	B	R	b	r	Ü	¿	...	↑
3	#	3	C	S	c	s	ä	Å	...	↑
4	\$	4	D	T	d	t	ö	Æ	...	↑
5	%	5	E	U	e	u	ü	φ	...	↑
6	&	6	F	V	f	v	ß	Á	...	↑
7	'	7	G	W	g	w	à	æ	...	↑
8	(8	H	X	h	x	°	ø	l	r
9)	9	I	Y	i	y	ç	á	l	↑
A	*	:	J	Z	j	z	é	é	l	↑
B	+	;	K	[k	[ù	ó	l	↑
C	,	<	L	\	l	l	è	á	l	↑
D	---	=	M]	m]	ó	ó	l	↑
E	_	>	N	^	n	^	í	í	l	↑
F	/	?	O	_	o	O	Ë		+	↑

- After programming has been completed, the programming mode is exited by pressing >Prog.< again. If – during programming of the name – no key will be pressed for more than one 1 min, the programming mode will be exited automatically.
- A log is printed out by pressing the key >E<. Check the data that has been entered.

8 Exchanging the ribbon

If the quality of the log printouts decreases, the ribbon must be exchanged.

- Remove the printer from the Prestomat.
- Loosen the two screws at the front of the printer and remove the housing element.
- Remove the old ribbon by pressing onto the right side of the ribbon (E) cartridge.
- In order to insert the new ribbon, thread in the right side first and then the left side. The paper must be between the ribbon and the cartridge.
- Close the housing and attach the printer to the Prestomat again.

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DeguDent GmbH