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# Multivac<sup>®</sup> compact

Instructions for use  
Mode d'emploi  
Istruzioni per l'uso

**DeguDent**

*A Dentsply International Company*

# Inhaltsverzeichnis / Content

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## 1 Working with the Multivac compact

### 1.1 Scope of delivery

- 1 basic unit Multivac compact (wall or table unit)
- 1 power cord
- 1 small mixing bowl with cover and stirring mechanism
- 1 large mixing bowl with cover and stirring mechanism
- 1 drilling template for mounting to the wall and mounting material (only for wall unit)
- 5 spare filters
- 1 instruction manual
- 1 guarantee card

### 1.2 Accessories available

<b>Items</b>	<b>REF</b>
Mixing bowl, extra-small with cover and stirring mechanism	5314 0113
Mixing bowl, small with cover and stirring mechanism	5314 0100
Mixing bowl, large with cover and stirring mechanism	5314 0101
Mixing bowl, extra-large with cover and stirring mechanism	5314 0114
Mixing bowl, extra-small	5351 1000
Mixing bowl, small	5351 1100
Mixing bowl, large	5351 1200
Mixing bowl, extra-large	5351 1300
Cover with stirring mechanism for mixing bowl extra-small	5351 4000
Cover with stirring mechanism for mixing bowl small	5351 4100
Cover with stirring mechanism for mixing bowl large	5351 4200
Cover with stirring mechanism for mixing bowl extra-large	5351 4300
Spare filter	5314 0105
Windows <sup>®</sup> PC program for programming the unit	5314 0102
PC connecting cable with adapter	5314 0103
Wall holder for mounting in front of cable ducts	5314 0104
Table stand (only for wall unit)	5314 0106

### 1.3 Technical data

Rated voltage:	see typeplate
Maximum power consumption:	300 W
Required fuse protection (200V - 240V):	2 AT
Required fuse protection (100V - 120V):	4 AT
Operating temperatures:	4 °C to 40 °C
Dimensions of wall unit (mm):	520 (h) x 385 (w) x 320 (d)
Dimensions of table unit (mm):	520 (h) x 385 (w) x 320 (d)
Weight of wall unit:	14.5 kg
Weight of table unit:	16 kg

## 1.4 For your personal safety

Please read this instruction manual carefully prior to the installation and initial operation. The information contained in this instruction manual must be observed.

The Multivac compact must only be operated with original spare parts and accessories. Only in this way the given performance data can be achieved and the required operational reliability is ensured.

Do not open the unit. Hazardous voltages have been applied in the interior of the unit. Maintenance work and repairs must only be performed by authorized service technicians.

Protect the unit against moisture to avoid short-circuits and damage to the unit.

Unplug the unit if it is not used for a longer period.

## 1.5 General information for the user


### 1.5.1 Proper use

- stirring of modelling and investment materials under vacuum
- mixing of silicone duplicating materials and alginates.

### 1.5.2 Special performance features

- reproduceable mixing results
- free programming of mixing speed, direction of rotation, vacuum and stirring time
- omission of manual premixing with spatula is possible
- microprocessor control

### 1.5.3 Control elements

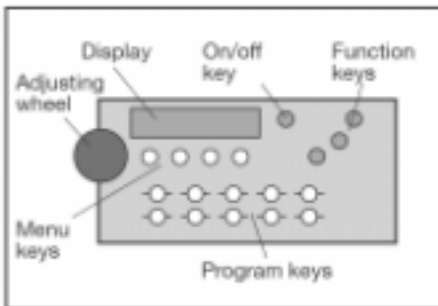#### On/off key

Activates the unit or switches it to a standby mode. The readiness for working of the Multivac compact is indicated in the display.

#### Display

Indicates the operating states of the Multivac compact.

#### Program keys

Allow to recall stored stirring programs.

## Function keys

Allow to switch on or switch off the motor and the vacuum pump in the manual operation mode as well to start and to cancel stirring programs in the program-controlled stirring mode.

## Menu keys


Allow to set data for stirring time, speed, direction of rotation and vacuum in the manual operation mode.

## Adjusting wheel

Allows to change data that have already been set. By pressing the adjusting wheel the changed data are confirmed.

### 1.5.4 Display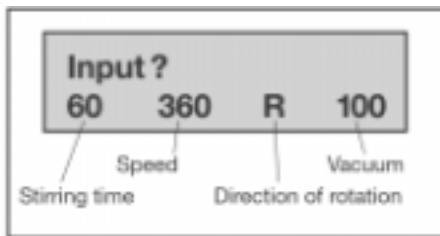

<b>Stirring time:</b>	1 s ... 240 s.
<b>Speed:</b>	100 rpm ... 600 rpm.
<b>Direction of rotation:</b>	left-hand – right-hand rotation
<b>Vacuum:</b>	20 % ... 100 %.

The basic settings for stirring time, speed, direction of rotation and vacuum that are displayed when the unit is switched on can be changed in the programming mode (section 3.1).

## 2 Putting into operation

### 2.1 Installation of the unit

#### Table unit:

- Place the Multivac compact on a stable, level surface.
- Connect the unit with the power supply system.

The connection for the power cord is on the right side in the rear part of the unit.

#### Wall unit:

- Insert the 3 fixation dowels into the wall according to the enclosed drilling template.

Make sure that there are no electricity lines or water pipes in the area of the drill holes.

- Screw the upper two fixation screws into the dowels.
- Hang in the Multivac compact.

- **Protect the unit against sliding upwards by screwing in the third fixation screw in the area of the column.**
- **Connect the unit with the power supply system.**

The connection for the power cord is on the right side at the rear part of the unit.

For mounting the unit in front of a cable duct a special wall holder (**REF 5314 0104**) is available.

## 2.2 Coupling in the mixing bowl photo

- **The bowl support must be brought to the lower position by pressing the unlocking button.**
- **Place the bowl into the center of the bowl support.**
- ⚠ Please ensure that the guide bar of the bowl support fully reaches into the groove of the bowl.
- **Slide the bowl support with the bowl fully upwards.**  
If the coupling has fully clicked into place, the sealing lip of the coupling lies on the cover of the bowl. If the preselected vacuum is not reached during the operation, an acoustic signal can be heard. In this case the bowl should be pressed against the coupling once more.

In order to facilitate coupling-in of the stirring blade into the drive, the drive is automatically switched on for a short time when the bowl support is lifted.

## 2.3 Manual operation

In the manual operation mode the stirring mechanism and the vacuum pump can be switched manually. The given values for stirring time, speed, direction of rotation and vacuum level can be adjusted.

- Switch on the unit. 
- **Select the given value to be changed.**  
The value to be changed is shown in the first line of the display.    
- **Turn the adjusting wheel until the desired value is shown in the display.**

- **Press the adjusting wheel to accept the value as a presetting.**

The changed value is now accepted in the lower line of the display.

- **Couple in the mixing bowl.**

- **Start the vacuum pump.** 

In the display a progress bar can be seen which indicates the current vacuum value. The vacuum is adjusted to the set value. The current vacuum is indicated in the lower display line.

- **Start the stirring mechanism.** 

In the display a progress bar can be seen which indicates the stirring time that has already run down. The current vacuum is indicated in the lower display line. During the progress, the stirring mechanism and the vacuum can be switched off or on at any time without influencing the chronological progress.

The stirring process can be interrupted at any time by pressing the cancellation key .

**After the set stirring time has expired, an acoustic signal can be heard.** The stirring process is stopped and the mixing bowl is ventilated automatically.

- **Press the unlatching button of the bowl support and bring the bowl support into the lower position again.**

The adjusted values are set back to the standard values after each stirring process.

If - during evacuating - a leak is found in the system, a warning signal can be heard and the displayed vacuum value is marked with an asterisk.

## 2.4 Starting of stirring programs

- **Switch on the unit.** 

- **Select a preset program.**  
The stored program name is shown in the display. The program selection can be changed by pressing a different program key or by turning the adjusting wheel.

- Couple in the mixing bowl.

- **Start the mixing program.** 

Depending on the program progress bar messages are shown in the display. The lower display line shows the current settings. The stirring process can be interrupted at any time by pressing the function key .


The stirring process is stopped and the mixing bowl is ventilated automatically. **After the end of stirring program an acoustic signal can be heard.**

- Press the unlatching button of the bowl support and bring the bowl support into the lower position again.

If - during evacuating - a leak is found in the system, a warning signal can be heard and an asterisk is shown in front of the displayed vacuum value.

### 3 Programming

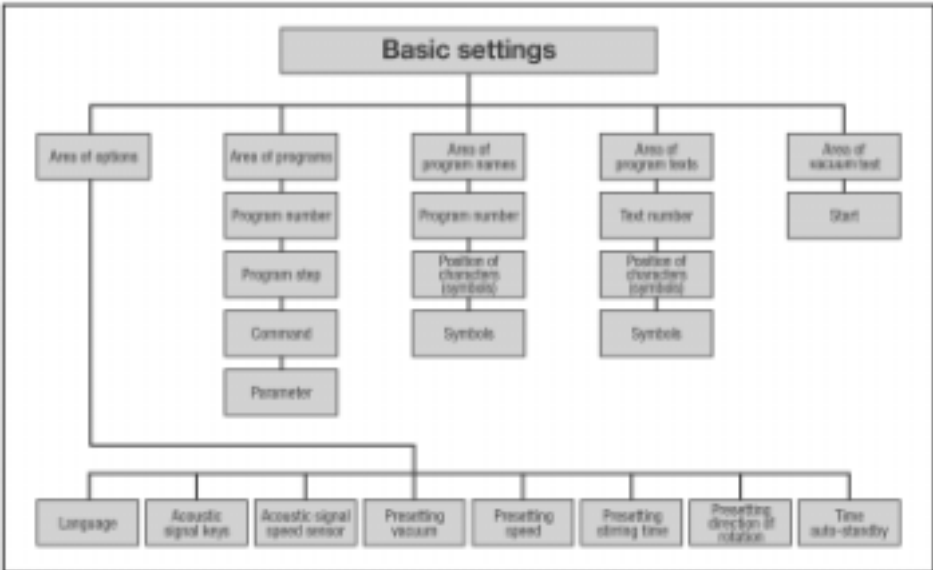#### 3.1 Programming mode

In order to enter the “programming” mode:

- Switch off the unit. 
- Press the program selection key and keep on holding it while switching on the Multivac compact.  + 

You are now in the “programming” mode. The selected “options” area is shown in the display.

- Turn the adjusting wheel until the desired area is shown in the display. The following areas are available: options, programs, program names, program texts, vacuum test.
- Press the adjusting wheel to enter the selected area.
- By pressing the cancellation key  you can leave an area at any time. By repeated pressing you will leave the programming mode and enter the operating mode of the unit.

## 3.2 Options

In this area the individual presettings can be changed.

- **Go to the “programming” mode (see section 3.1).**
- **Select the “options” area.**
- **Press adjusting button.**
- **Turn the adjusting wheel until the desired setting is shown in the display.**

You can select the following settings:

<b>Language</b>	(German, English, Spanish, Italian, French)
<b>acoustic signal</b>	keys (on, off)
<b>acoustic signal</b>	turn (on, off)
<b>presetting vacuum</b>	for manual operation
<b>presetting speed</b>	for manual operation
<b>presetting stirring time</b>	for manual operation
<b>presetting direction of rotation</b>	for manual operation

- **Press the adjusting wheel to change a setting.**
- **Turn the adjusting wheel until the desired setting is shown in the display.**
- **Press the adjusting wheel to accept the change.**

By pressing the cancellation key  a process can be terminated at any time and you will enter into the next (higher) control level.

## 3.3 Programs

In this area individual stirring programs can be changed or created.

- **Go to the “programming” operation mode (see section 3.1).**
- **Select the “programs” area.**
- **Press adjusting button.**
- **Turn the adjusting wheel until the desired program number is shown in the display.**
- **Press the adjusting wheel to change the selected program.**

- Turn the adjusting wheel until the desired program step is shown in the display.
- Press the adjusting wheel to change the desired program step.
- Turn the adjusting wheel until the desired program command is shown in the display.
- Press the adjusting wheel to accept the selected program command.  
Some commands require the additional selection of a corresponding parameter. If, e.g., the command “Motor rechts drehen (right-hand rotation of motor)” is selected, it is also required to enter the speed.

By pressing the  cancellation key a process can be terminated at any time and you will enter into the next (higher) control level.

#### The commands that can be selected and the corresponding parameters:

No	Command	Name	Parameter
0	End	End	-
1	Wait (maintain current status)	Wait	Time in sec (0..255)
2	Wait for key	Wkey	0 = “continue ?” 1..15 =text number of own texts 16 = acoustic signal key stroke 17 = acoustic signal program end 18 = acoustic signal error message 19 = acoustic signal speed sensor 20 = blank line
3	Wait for vacuum	Wvac	Vacuum in % (0..100)
4	Set vacuum without waiting	Vacu	Vacuum in % (0..100)
5	Right-hand rotation of motor	Right	Speed in rpm (0,100..600)
6	Left-hand rotation of motor	Left	Speed in rpm (0,100..600)
7	Output	Outpu	0 = default text with time bar 1..20 see “wait for key”
8	Presetting vacuum	Pvac	Vacuum in % (0..100)
9	Presetting speed	Pspee	Speed in rpm (0,100..600)
10	Presetting direction of rotation	Pdire	0=left-hand, 1=right-hand rotation
11	Presetting stirring time	Ptime	Time in seconds (0..240)
12	No action	Blank	-

To facilitate the creation of programs a Windows® PC program (**REF 5314 0102**) is available as accessory.

### 3.4 Program names

In this area the names of the stored programs can be changed.

- Go to the “programming” operation mode (see section 3.1)
- Select the “program names” area.

- **Press adjusting button.**
- **Turn the adjusting wheel until the desired program name is shown in the display.**
- **Press the adjusting wheel to change the selected program name.**
- **Turn the adjusting wheel until the desired letter position is shown in the display.**
- **Press the adjusting wheel to change the selected letter.**
- **Turn the adjusting wheel until the desired letter is shown in the display.**
- **Press the adjusting wheel to accept the selected letter.**

By pressing the cancellation key  a process can be terminated at any time and you will enter into the next (higher) control level.

### 3.5 Program texts

In this area texts that are displayed during the program course can be changed.

- **Go to the “programming” operation mode (see section 3.1).**
- **Select the “program texts” area.**
- **Press adjusting button.**
- **Turn the adjusting wheel until the desired program text is shown in the display.**
- **Press the adjusting wheel to change the selected program text.**
- **Turn the adjusting wheel until the desired letter position is selected in the display.**
- **Press the adjusting wheel to change the selected letter.**
- **Turn the adjusting wheel until the desired letter is shown in the display.**
- **Press the adjusting wheel to accept the selected letter.**

By pressing the cancellation key  a process can be terminated at any time and you will enter into the next (higher) control level.

## 3.6 Vacuum test

Start a test run to check the complete vacuum system.

- **Couple a large mixing bowl with stirring mechanism in the unit.**
- **Start the test run by pressing the adjusting wheel.**

After the test run passing of the test is confirmed with the display of "OK" (accepted), failing of the test with the display of "NOK" (not accepted).

**Please note that only a test run with a large mixing bowl produces correct test results.**

# 4 Maintenance and care

## 4.1 General

Maintenance and care include work which ensure perfect operational reliability of the Multivac compact.

- **Check the unit for mechanical damage in regular intervals. In this way accidents and unit breakdowns can be prevented.**
- **Immediately remove contaminations from all movable components. Only in this way smooth running and proper function of these components are ensured.**

The motor of the stirring mechanism and the vacuum pump are maintenance-free; the use of vacuum oil is not required.

**Prior to cleaning the unit must be switched off and unplugged.**

**When cleaning the unit make sure that no moisture can penetrate into the unit. In this way you avoid short-circuits which may result in the breakdown of the unit or in setting the unit on fire.**


Do not open the unit. Hazardous voltages have been applied in the unit. Repairs must only be performed by authorized service technicians.

Manipulations of the unit reduce the operational reliability and include the risk of damage to persons or materials.

## 4.2 Changing the filter

The filter must be changed if there is a considerable increase in the times for evacuating and ventilating of the mixing bowls or if the Multivac compact requests you to change the filter.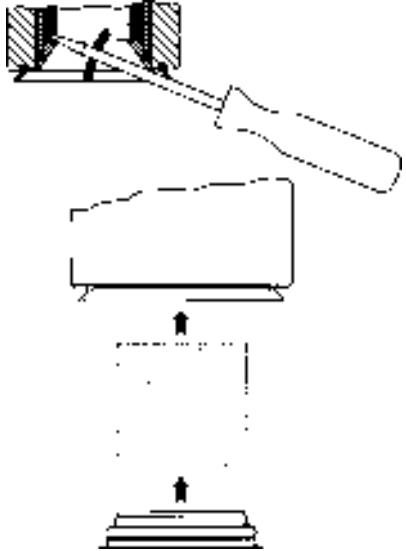

- **Remove the lock ring from the coupling with a slotted screwdriver.**
- **Pull out the old filter element.**
- **Remove adhering material residues from the coupling area.**
- **Place the new filter element onto the lock ring.**
- **Insert the filter element with lock ring into the coupling again.**  
The lock ring must engage again.

**Using the unit without filter may cause a damage of the vacuum system.**

## 5 Elimination of failures

### 5.1 Release of the fuse

**During switching on or during the start of the unit the internal unit fuse is released.**

The unit fuse is located next to the power supply connection. Repeated release of the fuse indicates a defect which must be eliminated by a service technician. Single release of the fuse can also be caused by jamming of the motor of the stirring mechanism. In this case - after eliminating the jamming - the fuse can be changed and the Multivac compact can be used again.

**The unit must be unplugged before the fuse is changed.**

## 5.2 Error messages

**During the program course an acoustic warning signal can be heard and the displayed vacuum value is marked by an asterisk.**

The set value is not achieved. Check whether leakage of the cover exists and for this purpose - press the mixing bowl against the coupling once more. If the same error occurs during the use of a mixing bowl / cover, the mixing bowl / cover must be exchanged.

Check if the seal in the coupling is clean and undamaged.

**After the end of the program the error message “Error vacuum” is displayed.**

During the stirring process the set vacuum has not been achieved. The stirring result may be unsatisfactory. Confirm the error message by pressing the <Start/Stop> key.

**After the end of the program the error message “Filter!” is displayed.**

The filter element is clogged. Change the filter element as described in section 4.2.

**During stirring operation the error message “Error overload” is displayed and the stirring mechanism stops.**

The motor of the stirring mechanism is overloaded. Check smooth running of the stirring mechanism and the drive system.

## 6 Statement of conformity

EU statement of conformity in accordance with EC guideline for machines 89/392/EWG

DeguDent GmbH

P.O. Box 1384  
D-63403 Hanau

This is to certify the conformity of the product:

### Multivac® compact/ stirring and evacuating unit

with the basic requirements of the following EC guidelines:

1. EC guideline for machines 89/392/EEC with 1. Modification 91/368/EEC, 2. Modification (93/44/EEC) and 3. Modification 93/68/EEC
2. EC guideline for electrical equipment to be used within certain voltage limits (Guideline for Low Voltage), 73/23/EEC with 1. Modification 93/68/EEC
3. EC guideline on electromagnetic compatibility 89/336/EEC with 1. Modification 93/68/EEC

The following harmonized standards were applied:

EN 292-1: 1991

EN 292-2: 1991


EN 60204-1: 1992

EN 61010-1: 1993

EN 50081-1: 1991

EN 50082-1: 1991

The procedure was adhered to in accordance with Appendix V of the guideline for machines.

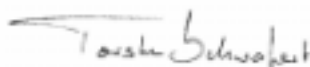Torsten Schwafert  
Director business area  
dental laboratory products

Dr. Alexander Völcker  
Research & Development

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