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Multimat2 Touch

Multimat2 Touch+Press

CZ	Krátký návod k použití pro přístroje Multimat2 Touch a Multimat2 Touch+Press	4
D	Kurzbedienungsanleitung für die Geräte Multimat2 Touch und Multimat2 Touch+Press	5
DK	Forenklet brugsanvisning til Multimat2 Touch og Multimat2 Touch+Press	6
E	Instrucciones de manejo abreviadas para los aparatos Multimat2 Touch y Multimat2 Touch+Press	7
F	Mode d'emploi abrégé pour les appareils Multimat2 Touch et Multimat2 Touch+Press	8
FIN	Multimat2 Touch ja Multimat2 Touch+Press laitteiden pikakäyttöohje	9
GB	Short Operating Instructions for Multimat2 Touch and Multimat2 Touch+Press	10
GR	Σύντομες οδηγίες χρήσης για τις συσκευές Multimat2 Touch και Multimat2 Touch+Press	11
H	Rövid kezelési útmutató a Multimat2 Touch és Multimat2 Touch+Press készülékekhez	12
I	Brevi istruzioni per l'uso per gli apparecchi Multimat2 Touch e Multimat2 Touch+Press	13
LT	Trumpas vartotojo žinynas prietaisams Multimat2 Touch ir Multimat2 Touch+Press	14
LV	Īsa lietošanas pamācība ierīcei Multimat2 Touch und ierīcei Multimat2 Touch+Press	15
N	Forenklet bruksanvisning til apparatene Multimat2 Touch og Multimat2 Touch+Press	16
NL	Korte gebruiksaanwijzing voor de apparaten Multimat2 Touch en Multimat2 Touch+Press	17
P	Instruções de uso abreviadas para os aparelhos Multimat2 Touch e Multimat2 Touch+Press	18
PL	Skrócona instrukcja obsługi dla urządzeń Multimat2 Touch i Multimat2 Touch+Press	19
S	Snabbguide för apparaterna Multimat2 Touch och Multimat2 Touch+Press	20
SK	Krátký návod na obsluhu pre prístroje Multimat2 Touch a Multimat2 Touch+Press	21
SLO	Kratko navodilo za uporabo naprav Multimat2 Touch in Multimat2 Touch+Press	22

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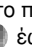
Short Operating Instructions for Multimat2 Touch and Multimat2 Touch+Press

These Short Operating Instructions are designed to simplify your daily work with Multimat2 Touch and Multimat2 Touch+Press. It cannot serve as a substitute for the detailed User Manual. Consult the full manual whenever you have any questions. These Short Operating Instructions address all program features up to and including program version V1.05.

Action	Step 1	Step 2	Step 3	Step 4	Step 5	Manual
1. Turning on/off	Close firing chamber (press  key)	Press On/Off key 				4.3
2. Opening/closing the chamber	Press  or  key					4.3
3. Running a program	Press  key	Enter program number				4.3
4. Programming	Select firing process	Select individual programs; "New" sensor field	Enter program name (A-Z)	Enter parameters (0-9)  	Press  key	5.1 ff
5. Storing a program	Press  key					
6. Starting/stopping a program	Press Start/Stop  key					
7. Copying a program	Press  key	Enter number of program to be copied	Press  key	Enter new program number	Press  key	5.8.5
8. Change firing parameters during or for a cycle	Press the sensor field for the parameter to be changed	Enter parameters (0-9)				
9. Defining menu settings	Press  key					
10. Turning off rapid cooling option	Press  key					
11. Activating/deactivating night mode	Press  key					
12. Confirming messages	Press  key					
13. Increasing/decreasing the display contrast	Press  sensor field until basic menu settings are shown	Press  and 				3.4.6.10
14. Showing all set points during the firing cycle	Press  sensor field					
15. Transferring data using Smart Media Cards	Press  key	Select configuration	Select Smart Media Cards	Insert Smart Media Cards	Select one of the three menus	6.16
16. Deleting a program	Press  key	Select Deleting a Program	Select one of the menus			4.5.4

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DeguDent GmbH