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# Multimat C

<b>D</b>	Kurzbedienungsanleitung für Ihren Multimat C .....	4
<b>DK</b>	Kort brugervejledning til Multimat C .....	5
<b>GB</b>	Quick Start Instructions for your Multimat C .....	6
<b>EST</b>	Multimat C lühikene kasutusjuhend .....	7
<b>FIN</b>	Lyhyt käyttöohje Multimat C uunille .....	8
<b>GR</b>	Σύντομες οδηγίες χρήσης για το Multimat C σας .....	9
<b>NL</b>	Korte handleiding voor de bediening van uw Multimat C .....	10
<b>I</b>	Istruzioni per l'uso in forma succinta per il vostro Multimat C .....	11
<b>LV</b>	Jūs Multimat C īsā lietošanas pamācība .....	12
<b>LT</b>	Trumpa Multimat C naudojimo instrukcija .....	13
<b>M</b>	Struzzjonijiet biex Tibda Malajr għall-Multimat C tiegħek .....	14
<b>PL</b>	Krótką instrukcja obsługi Multimat C .....	15
<b>P</b>	Manual de instruções abreviado para o seu Multimat C .....	16
<b>S</b>	Förkortad handledning för Multimat C .....	17
<b>SK</b>	Stručný návod na obsluhu pre Váš Multimat C .....	18
<b>SLO</b>	Kratka navodila za uporabo vaše naprave Multimat C .....	19
<b>E</b>	Instrucciones de servicio resumidas para el Multimat C .....	20
<b>CZ</b>	Krátký návod k pou ití pro váš Multimat C .....	21
<b>H</b>	Multimat C rövid kezelési utasítása .....	22

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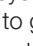
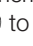
# Quick Start Instructions for your Multimat C

Action	Step 1	Step 2	Step 3	Step 4	Step 5
<b>1.</b> Switching on or off	Press key 				
<b>2.</b> Lift up/down	Press keys  				
<b>3.</b> Recall program	Press key 	Use numeric keypad  -  to enter program number			
<b>4.</b> Programming	Press key 	Use cursor keys     to go to firing parameters	Use numeric keypad  -  to change firing parameters		
<b>5.</b> Store program	Press key 	Use numeric keypad  -  to enter program number			
<b>6.</b> Start/cancel program	Press key 				
<b>7.</b> Copy program	Press key 	Use numeric keypad  -  to enter program number you want to copy	Press key 	Use numeric keypad  -  to enter desired program number	
<b>8.</b> Change parameters during firing process or for one off use	Press key 	Use cursor keys     to go to firing parameters	Use numeric keypad  -  to change firing parameters	Press key 	
<b>9.</b> Cancel vacuum	Press key 				
<b>10.</b> Menu settings	Press key 	Use cursor keys   to select submenu and press key 	Use cursor keys   to select parameters	Use cursor keys   to change values	Press key  as often as necessary until program reappears
<b>11.</b> Activate quick cooling	Press lift key  with the chamber open				
<b>12.</b> Deactivate quick cooling	Press key 				
<b>13.</b> Confirm information messages	Press key 				
<b>14.</b> Increase display contrast	Press key  and hold,	and press key  to increase contrast			
<b>15.</b> Decrease display contrast	Press key  and hold,	and press key  to decrease contrast			
<b>16.</b> Display all nominal values while firing is in progress	Press key  until nominal values appear				

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DeguDent GmbH