DENTSPLY
TRUBYTE

Lucitone 199[®]

Denture Base Resin

Prothesenkunststoff

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Resina per basi protesiche

Resina para bases de dentaduras

DIRECTIONS FOR USE
GEBRAUCHSANWEISUNG
MODE D'EMPLOI
INDICAZIONI PER L'USO
INSTRUCCIONES DE USO

CE

Caution: U.S. Federal Law restricts this device to sale by, or on the order of, a Dentist.

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Indications for Use: LUCITONE 199 Resin is indicated for the fabrication of prosthodontic appliances.

Contraindications:

1. LUCITONE 199 Resin is contraindicated for patients and users with a history of allergic reaction to methyl methacrylate monomer.

Warnings:

1. LUCITONE 199 Resin contains polymerizable monomers which may cause skin sensitization (allergic contact dermatitis) or other allergic reactions in susceptible persons. Wash thoroughly with soap and water after contact. If skin sensitization occurs, discontinue use. If dermatitis or other symptoms persist, seek medical advice.
2. Avoid inhalation or ingestion. High vapor concentration can induce headache, and irritation of eyes and respiratory system. Liquid contact with eyes may cause possible corneal damage. Excessive long-term exposure may be associated with other more serious health effects. Monitor air quality per OSHA standards.

Inhalation: Move subject to fresh air. Give oxygen or artificial respiration as required.

Ingestion: Contact your regional Poison Control Center immediately.

Eye Contact: Flush eyes promptly with copious amounts of water for 15 minutes, and consult a physician. Wash skin with soap and water.

Precautions:

1. Users with special skin problems, cuts, or abrasions should wear protective gloves.
2. When grinding prosthodontic resins, proper ventilation, masks, and vacuum systems should be used.
3. Store at 60°-80°F (15°C - 26°C) away from moisture and direct sunlight. The liquid contains unsaturated monomers which may polymerize prematurely if stored at excessively high temperature and/or in sunlight. The powder contains an organic polymerization initiator which may degrade if stored at excessively high temperature.

4. LUCITONE Liquid and LUCITONE 199 Resin uncured resin are hazardous materials. Dispose in accordance with Federal, State, and local regulations.
5. LUCITONE Liquid contains methyl methacrylate monomer, a flammable liquid with a flash point of 10°C (50°F). Keep away from heat, sparks, and open flame.
6. Use liquid in a well ventilated area. Replace cap when not in use.
7. Do not pack denture resin after work time has elapsed or material has become rubbery.
8. Use of wax solvents is not recommended. Residual solvent may cause poor bonding of teeth to acrylic base.
9. Corneal damage, headache, nausea, vomiting may occur with exposure to methyl methacrylate monomer. (See Warnings and Precautions statements.)
10. Allergic contact dermatitis and other allergic reactions may occur in susceptible individuals. Residual monomer in fully cured materials can be minimized by soaking the cured prosthesis in warm water for several days.
11. Particulates will be generated when grinding acrylic resins. Eye, skin and respiratory irritation may occur if appropriate engineering controls are not used.

Type and Class (according to ADA Spec. 12)	Type 1, Class 1
Storage temperature for powder and liquid	60-80°F (15-26°C)
Powder/Liquid ratio	21g (32 cc)/10 ml
Mixing time (time required to wet all particles)	15-30 seconds
Time to reach packing plasticity @ 23 ± 1°C	9±2 minutes
Working time	10 ± 4 minutes
Material used to prepare mold	gypsum
Temperature of mold when packing	room temperature to 110°F (43°C)
Recommended cure time and temperature	
1st stage	1-1/2 hours @ 163°F (72°C)
2nd stage	1/2 hour @ 212°F (100°C)
Alternate cure time and temperature	9 hours @ 163°F (72°C)
Method of cooling flask, time and temperature	
1st stage	1/2 hour in air at 60-80°F (15-26°C)
2nd stage	1/4 hour in water at 60-80°F (15-26°C)

STEP BY STEP INSTRUCTIONS

PROPERTIES AND RECOMMENDED PROCEDURES - The preceding table describes typical properties of LUCITONE 199 Resin, plus recommended temperatures and other important processing information. Further amplification of this condensed information is found later in the text.


FLASKING - Use conventional detail compression molding methods and gypsum materials for flasking.

WAX ELIMINATION - Soften wax in boiling water for approximately 6 minutes. Separate flask and remove wax by flushing with a solution of boiling water to which a detergent has been added. Use clean boiling water for final flush. **DO NOT USE WAX SOLVENTS.** Be sure case is free of wax and grease.

TINFOIL SUBSTITUTE - Apply AL-COTE® Separator liberally to areas of the warm mold (approx. 120°F/48°C) that will contact the resin. Remove excess AL-COTE film with a dry brush. (AL-COTE Separator does not form a film on teeth.) The film of AL-COTE Separator should be thoroughly dry (approx. 3

minutes following application) before the resin is packed.


IMPORTANT - Measure powder and liquid accurately. Do not whip air into mix by excessive spatulation.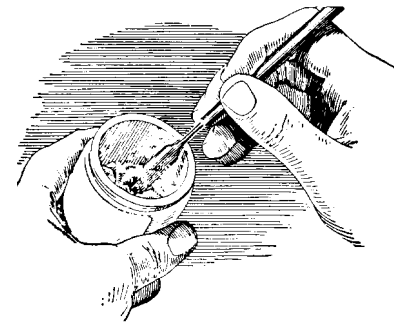


MIXING - To assure color uniformity, shake powder jar (with a rocking motion) to prevent concentration of small particles at the bottom of jar. Add 1 unit powder 21 g (32cc) to 10 ml liquid. Stir sufficiently (15 seconds) to assure wetting of all powder particles. Do not overmix. Cover mixing jar and allow material to reach packing consistency (approx. 9 minutes at room temperature of 73° ± 2°F). (22° ± 1°C)

Powder measure and liquid graduate are supplied in packages of LUCITONE 199 Resin. The following table provides the correct ratios for various cases.

CASE	POWDER	LIQUID
Extra Large	42 cc	13 ml
Large	32 cc	10 ml
Average	26 cc	8 ml
Partial	19 cc	6 ml

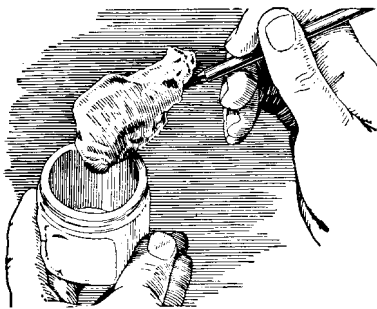
PACKING - LUCITONE 199 Resin should be packed in warm flask (room temperature to 110°F/43°C). The correct packing consistency is identified when the resin dough is free from stickiness but not rubbery. Remove resin dough from jar and condense with finger pressure into mold.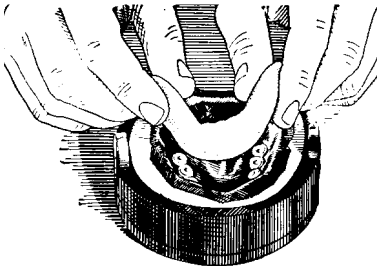

TRIAL PACKING - LUCITONE 199 Resin does not require trial packing except in extra-thick cases. However, trial packing may be employed if the operator desires. The flask should be closed **slowly** in all cases.

If the no trial packing technique is employed, it is essential that a sufficient amount of material be used to assure a full mould. Excess resin squeezing out of the complete periphery of the two halves of the flask during closure is a good indication of sufficient volume of material.

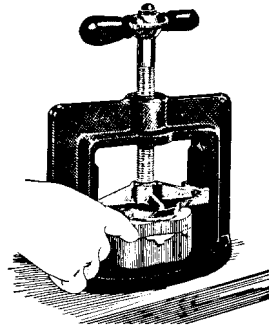Close flask slowly

WORK TIME - The time of LUCITONE 199 Resin is approximately 10 minutes. Never pack after work time has elapsed or material has become rubbery.

CURING - Submerge closed flask (locked by compress or spring clamp) in water at 163° ± 2°F/72°C ± 1°C for 1 1/2 hours. Follow by 1/2 hour in boiling water. A periodic check of water

bath temperature with an accurate thermometer is recommended. Alternate cure: 9 hours in water bath at 163° ± 2°F/72°C ± 1°C. No boil necessary.

CURING OF EXTREMELY THICK CASES - Extremely thick cases require special handling to prevent porosity and excessive surface shrinkage. The following procedure should be employed:

1. Pack material when it reaches a heavy, putty-like consistency (approx. 12 minutes at 73° ± 2°F/23°C ± 1°C).
2. Condense well during packing by closing flask **slowly** and trial packing at least 3 times.
3. Closed flask should remain under pressure for 30 minutes prior to curing.

4. Use alternate cure method as specified.

COOLING - Bench cool the clamped flask at room temperature for 30 minutes. Then immerse in cool water (60-80°F/15-26°C) for 15 minutes before deflasking.

FINISHING AND POLISHING - The denture should be finished and polished in the usual manner.

REPAIRING AND RELINING - Relines or repairs to the completed denture may be made with self-curing DENTSPLY® REPAIR MATERIAL or LUCITONE 199 REPAIR MATERIAL in accordance with directions provided. If desired, repairing or relining can also be done by the usual technique using LUCITONE 199 Resin itself. Cure in water at 163°F (72°C) for 9 hours - do not boil.

MSDS for U.S. Market Only.

MATERIAL SAFETY DATA SHEET

FORM NO.: 150

SECTION I - GENERAL INFORMATION

1.1 Product Trade Name: Lucitone 199® Powder

1.2 Part (Item) Number: 688103, 688203, 688303, 688403, 688105, 688205, 688305, 688405, 688106, 688206, 688306, 688406, 688111, 688211, 688311, 688411, 688102, 688107, 688112, 688113, 688114, 688115, 688120, 688220, 688320, 688420

1.3 Chemical Name: Polymethylmethacrylate

1.4 Synonyms:

1.5 Division Name: Dentsply/Trubyte Division
Address: 570 W. College Ave.
City, State, Zip: York, PA 17405-0872

1.6 Phone No. for Info.: 717-845-7511

Dentsply
"FOR CHEMICAL EMERGENCY"

SPILL, LEAK, FIRE, EXPOSURE OR ACCIDENT
CALL CHEMTREC - DAY OR NIGHT

1.7 Emergency Phone No.: 800-424-9300 1.12 National Stock No.: N.A.
Chemtrec 1.13 Specification No.: N.A.
1.8 Date: 3/27/00 1.14 Chemical Family: Acrylic Polymer
1.9 Division Contact: Scott E. Shaffer
1.10 Signature: *Scott E. Shaffer* 1.15 Type/Grade/Class: N.A.
1.16 NRC/EPA Number: N.A.

1.11 Formula: $(CH_2(CH_3)COOCH_3)_n$

SECTION II - HAZARDOUS INGREDIENTS

CAS Number	Chemical Name (Ingredients)	%	TLV mg/m ³	OSHA PEL
94-36-0	Benzoyl Peroxide	<.2	5.0	

SECTION III - PHYSICAL PROPERTIES

3.1 Boiling Point: N.A. 3.11 Specific Gravity: N.E.

3.2 Vapor Pressure: N.A. 3.12 pH: N.A.

3.3 Vapor Density: N.A. 3.13 Evaporation Rate: N.A.

3.4 Critical Temp: N.E. 3.14 Viscosity: N.A.

3.5 Decomposition Temp.: 200°C/392°F 3.15 % Volatile by Vol.: 0.0

3.6 Melting/Freezing Pt.: N.A. 3.16 Magnetism: N.A.

3.7 Solubility in Water: Not Soluble 3.17 Autoignition Temp.: N.E.

3.8 Critical Pressure: N.A. 3.18 Corrosion Rate: N.A.

3.9 Perm. Exp. Limit: N.A. 3.19 Molecular Weight: N.A.

3.10 Appearance and Odor: Pink free flowing powder with faint odor in bulk

SECTION IV - FIRE AND EXPLOSION HAZARD DATA

4.1 Flash Point (Method Used): 304°C(572°F) (TCC)

4.2 Flammable (Explosive) Limits:
LEL: 20g/m³ (dust cloud) UEL: N.A.

4.3 Extinguishing Media: Water spray, water foam, carbon dioxide, dry chemical.

4.4 Special Fire Fighting Procedures and Equipment: Avoid extinguishing methods which may generate dust clouds. Water stream can disperse dust in air, producing a fire hazard and possible explosion hazard if exposed to ignition source.

4.5 Unusual Fire and Explosion Hazards: Polymer dust is combustible but not easily ignited. The explosive limits of the polymer particles suspended in air are approximately those of coal dust. Fire fighters should avoid extinguishing methods which generate dust clouds. Fire Fighters should wear self-contained breathing apparatus.

MSDS for U.S. Market Only.

Lucitone 199® Powder, cont'd
SECTION V - HEALTH HAZARD DATA

- 5.1 Effects of Acute and Chronic Over Exposure: It is not known to cause significant health problems. It is considered an inert or nuisance dust. Avoid inhalation of dust. Keep dust out of eyes to prevent possible irritation.
- 5.2 Emergency and First Aid Procedures: INHALATION: Remove to fresh air. Get medical help if discomfort persists. EYES: Flush with water for 15 minutes, including under eyelids. SKIN: Wash with soap and water. INGESTION: Rinse mouth with water. Do not induce vomiting. Call doctor if amount was large.

SECTION VI - REACTIVITY DATA

- 6.1 Unstable: Stable: X
- 6.2 Conditions to Avoid (Stability): Heat above 200°C (392°F)
- 6.3 Incompatibility (Materials to Avoid): Strong oxidizing agents
- 6.4 Hazardous Decomposition Products: Methylmethacrylate and Carbon Monoxide
- 6.5 Hazardous Polymerization: May Occur: Will Not Occur: X
- 6.6 Conditions to Avoid (Polymerization): Will not polymerize.

SECTION VII - SPECIAL PRECAUTIONS AND SPILL OR LEAK PROCEDURES

- 7.1 Handling and Storage Precautions: Store in cool dry place. Keep container closed to prevent water absorption and contamination.
- 7.2 Other Precautions: Observe normal warehousing procedures. Store away from food and beverages.
- 7.3 Steps to be Taken in Case Material is Released or Spilled: Sweep up to avoid slipping hazard. Keep airborne particulate at a minimum when cleaning up spills.
- 7.4 Waste Disposal Procedures: May be disposed of in landfill or incinerated. Follow Federal, State, and Local regulations for disposal.
- 7.5 Neutralizing Agent: N.A.

SECTION VIII - OCCUPATIONAL PROTECTION MEASURES

- 8.1 Respiratory Protection: Nuisance dust type if needed. An approved dust mask should be worn if exposure to high levels of dust are likely.
- 8.2 Ventilation: Local exhaust at processing equipment
- 8.3 Protective Gloves: Not required.
- 8.4 Eye Protection: Safety glasses
- 8.5 Other Personal Protective Equipment: None.

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Marca Registrada - Chile, Guatemala, Mexico, The Trademark under which this product is sold is registered in various countries.
© 1984, 1986, 1994, 1997, 1999, 2002 DENTSPLY International Inc. All rights reserved.
DENTSPLY International Inc.
DENTSPLY DeTrey GmbH
Printed in U.S.A. 588001 Rev. D (3/02)